

LogiLink_®

Disposal Face Mask, 5 pcs FFP2, Foldable Earloop type

Art. No.: HC0002

FFP2 Disposal Face mask with earloop has effective protection against dust, haze, droplets and other particles.

Features:

- » Filtering at least 95% of dust, haze, droplets and other particles
- » 4-layer purification: Melt-blown Nonwovens, moderate thickness for particle absorption, double layered electrostatic filter, and highly efficient at filtering particles
- » Advanced nonwovens fabric provides easier breathing
- » Comfortable to wear, low impedance filter material with no odor
- » Adjustable metal strip ensures optimal sealing between the bridge of the nose and the mask
- » Stanard: FFP2 NR EN 149:2001+A1:2009
- » 5 pieces per package

Instructions:

- 1. Unfold the mask and pull the bands outward on each end.
- 2. Position the mask on your face with the adjustable metal strip facing up.
- 3. Place the bands around both ears.
- 4. Press down lightly on the metal strip to ensure the mask is securely sealed.









Warning:

Do not use the mask in medical and surgical environments.

Do not use the mask in an environment where the oxygen concentration is below 19.5%.

Do not use the mask if you have a beard or other facial hair that has direct contact between your face and the edge of the mask. Do not use the mask in an environment of toxic gas.

Note:

This mask does not completely eliminate the risk of all diseases and infections. The mask must be sealed properly in order to be effective, misuse can result in a higher risk of contracting illness. For adult use only.

Store in a dry area at room temperature and do not wash the mask with water.









LogiLink_®

Disposal Face Mask, 5 pcs
FFP2, Foldable Earloop type

Art. No.: **HC0002**

Package Content:

» 1 x Disposal Face Mask, 5 pcs

Package Information:

» Packing Dimension: 251 x 160mm

» Packing Weight: kg

» Carton Dimension: 560 x 460 x 370mm

» Carton Q'ty: 200pcs» Carton Weight: 7.6kg









